
**Geratherm®**  
*active control+*

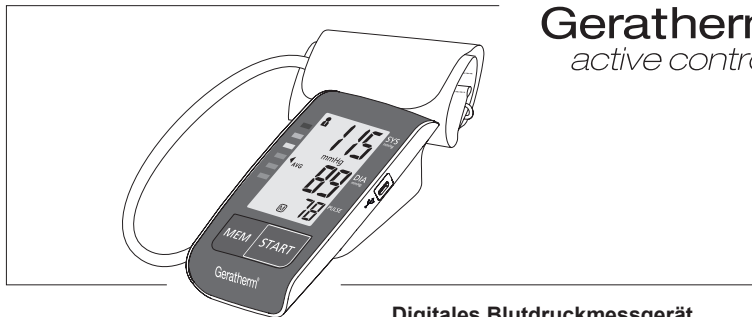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CE0197

# Digital Blood Pressure Monitor for the Upper Arm

**Geratherm<sup>®</sup>**  
*active control+*

EN

**INSTRUCTIONS FOR USE**

GT-1115

**CE0197**

## Contents

<b>General description</b> .....	31
<b>Warranty</b> .....	32
<b>Measurement Principle</b> .....	32
<b>Precautions</b> .....	32
<b>Cleaning</b> .....	32
<b>Maintenance and disposal</b> .....	35
<b>Storage</b> .....	36
<b>Information you should know before operating the unit</b> .....	36
<b>Description of the device</b>	
Information about the Device .....	38
Explanation of the Display .....	39
<b>How to operate the unit</b>	
Power Supply and Charge Power .....	40
Notes on rechargeable battery .....	42
Activate your blood pressure monitor .....	43
Setting Date and Time .....	43
Select User .....	44
Attaching the pressure cuff .....	44
Posture during measurement .....	45
<b>Functions</b>	
Measuring blood pressure .....	46
Irregular heartbeat detector .....	47
Retrieving stored readings.....	47
Deleting readings .....	49
<b>Care and maintenance</b> .....	50
<b>Error messages</b> .....	51
<b>Specifications</b> .....	52
<b>Quality standard</b> .....	53
<b>Symbol index</b> .....	53
<b>Information on electromagnetic compatibility (EMC)</b> .....	54

## General description

Thank you for choosing the Geratherm® active control+ blood pressure monitor.

**Before using your device for the first time, please read this instruction manual carefully.**

This blood pressure monitor uses the oscillometric method of measuring the systolic and diastolic blood pressure and the heart rate. The measurements are taken at the upper arm and all values appear on an LCD display screen.

This blood pressure monitor complies with the standards for medical devices. The blood pressure monitor can be used by adults with an arm circumference of 22 cm to 42 cm.

The Geratherm® active control+ is intended for the non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the arm or for any function other than measuring blood pressure.

The manufacturer cannot be held liable for any damage caused by improper use or failure to comply with the safety instructions in the instruction manual.

### **MEDICAL CONTRAINDICATIONS**

Consult a doctor before using the device if you are pregnant, have electrical device implants (e.g. pace maker, defibrillators), suffer from pre-eclampsia, atrial fibrillation, peripheral arterial diseases, have received intravascular therapy or had a mastectomy.

EN

Do not use the device if you are pregnant or suspect you may be pregnant. As well as the possibility of inaccurate measurement results, it is not known what effects the device might have on the foetus.

## Warranty

The warranty for this blood pressure monitor is valid for any error on the part of the manufacturer under normal use for 2 years from the date of purchase. If your unit does not function properly due to defective parts or assembly, we will repair it free of charge.

With the exception of the battery and cuff, all parts of the unit are subject to this warranty. Damage caused by improper handling of your unit is not guaranteed.

## Measurement Principle

This product uses the Oscillometric Measuring method to detect blood pressure. Before every measurement, the unit establishes a "zero pressure" equivalent to the air pressure. Then it starts inflating the cuff, meanwhile, the unit detects pressure oscillations generated by beatto-beat pulsatile, which is used to determine the systolic and diastolic pressure, and also pulse rate.

EN

## Precautions

- The device is intended for private use only.
- Before using the device, ensure that it functions safely and is in perfect condition. Check the device. Do not use the device if it is damaged in any way. Continuing to use a damaged device can lead to injuries, false readings and serious danger.
- These instructions and the device are no substitute for visiting your doctor. Neither the information contained here nor this device may be used to diagnose or treat health problems or to prescribe medicines. If you have a medical problem or suspect that you may have such a problem, please seek immediate advice from your doctor.
- Keep the device out of the reach of infants, children and pets. The inhalation or swallowing of small parts is dangerous and can be fatal. To avoid accidental strangulation, keep the product away from children and do not place the tube around your neck.
- Do not mistake self-monitoring for self-diagnosis. This device enables you to monitor your blood pressure. Do not undertake any therapeutic measures on the basis of measurements you have taken yourself.

## Precautions

- Under no circumstances should you change the dosage of the medication that your doctor has prescribed for you. Consult your doctor if you have any questions about your blood pressure.
- Wait 30 to 45 minutes before measuring if you have recently consumed a drink containing caffeine or have smoked a cigarette.
- Relax for at least 5 to 10 minutes before each measurement.
- If you experience any discomfort during measurement, such as pain in the arm or other complaints, press the 'START' button to deflate the cuff. Release the cuff and remove it from your arm.
- Measurements taken too frequently or consecutively can lead to disturbances in the blood circulation and injury.
- Avoid any kinking, crushing or constricting of the connecting tube of the cuff during use, as this could lead to a continuous increase in the cuff pressure, resulting in interruption of the blood flow and causing injury to the patient.
- Please use the device under the environmental conditions described in the instructions. Otherwise the performance and service life of the blood pressure monitor will be adversely affected.
- Please only use accessories that have been approved by the manufacturer. Otherwise, damage may be caused to the device, the user may suffer injury, or inaccurate measurements may occur.
- In order to avoid inaccurate measurements, do not expose the blood pressure monitor to strong electromagnetic fields, interference signals or electrical fast-transient/burst signals.
- In the event of an unexpected error message or problems with the product, please contact your retailer or Geratherm Medical AG.
- Please notify Geratherm Medical AG of any unexpected occurrences or incidents.
- The connector pins on the plug/adaptor insulate the device from the power supply. Do not place the device in a position that makes it difficult to disconnect it from the power supply in order to safely stop the operation of the device.
- The operator of the device must not touch the display unit, the battery/adaptor and the patient at the same time.
- Take care to avoid strangulation by cables and hoses, especially due to excess length.
- The materials used in the cuff have been tested and have been found to comply with the requirements of the standards ISO 10993-5:2009 and ISO 10993-10:2010. The fabrics used will not cause any potential irritation or allergic reaction.

## Precautions

- Do not use the cuff on damaged skin.
- Do not use the device if you are allergic to polyester or synthetic fabrics.
- Do not connect the air tube to any other medical equipment, as this can lead to high pressure or conduct air into intravascular systems, which may lead to injury.
- When the cuff pressure reaches 40 kPa (300 mmHg), the air will be automatically released. In the rare event of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately and press the 'START/STOP' button to end the measurement. Prolonged high pressure applied to the arm may result in ecchymosis. (Cuff pressure >299 mmHg or a constant pressure of >15 mmHg for longer than three minutes).
- This blood pressure monitor is not an AP/APG device and is not suitable for use in the presence of inflammable anaesthetic gases with air containing oxygen or nitrous oxide.
- Do not wrap the cuff around an arm in which an arteriovenous shunt has been implanted or to which other medical devices are attached, as this could lead to temporary functional failure of the medical devices being used.
- This device is not suitable for continuous monitoring during medical emergencies and surgery. This device cannot be used simultaneously with HR surgical devices.
- This device is not intended for patient transportation outside of a healthcare facility.
- Please ensure that the operation of the device does not permanently impair the patient's blood flow.
- If you need to test the measuring accuracy of the Geratherm® active control+, please contact the manufacturer or your retailer.
- Wireless devices such as home networks, mobile phones, cordless phones and their bases, or walkie-talkies can affect this device and should be kept at a minimum distance (d) from it. This distance is calculated by the MANUFACTURER from the column 80 MHz to 5.8 GHz of table 4 and table 9 of IEC 60601-1-2: 2014.

## Cleaning

- Please use a soft cloth for cleaning and do not use any abrasive or volatile cleaning agents. Use a solvent-free cleaner.
- Do not wash the cuff in the washing machine or dishwasher!
- The service life of the cuff can vary according to the frequency of washing, the condition of the skin and the manner of storage. The typical service life is approximately 10.000 measurements.
- Do not clean the blood pressure monitor while it is charging. Before cleaning, always remove it from the charger first.

## Maintenance and Disposal

- **Warning:** Do not carry out any maintenance or upkeep while the device is in operation.
- Dispose of the unit, accessories and components in accordance with local regulations.
- In the event of malfunction, do not attempt to repair the device yourself. Only permit repairs to be carried out by authorised technicians.
- Upon request, the manufacturer will provide circuit diagrams, parts lists, descriptions, calibration instructions, etc., to support service personnel in the repair of parts.
- The device does not need to be calibrated for two years after reliable maintenance.
- Do not dispose of your blood pressure monitor in fire. The rechargeable battery can explode and lead to injury or death. Batteries (battery pack or inserted batteries) must not be exposed to excessive heat such as sunshine, fire or similar.

EN

## Storage

- When it is not in use, please store the blood pressure monitor in a dry room and protect it from extreme humidity, heat, fluff, dust and direct sunlight. Do not place any heavy objects on top of the storage bag.
- This device contains sensitive parts and must be handled with care. Pay attention to the storage and operating instructions.
- It takes at least 30 minutes for the device to warm up from the minimum storage temperature to be ready for its intended use. The device must be allowed to cool down from the maximum storage temperature for at least 30 minutes before it is ready to be used again.
- Store the blood pressure monitor in a cool, dry and well-ventilated place. Keep it away from fire and heat sources, otherwise the battery could explode!

## Information you should know before operating the unit


### EN

#### What is blood pressure?

A force is created by the heart as the ventricle forcibly ejects blood into the blood vessels and through the vascular system. Another force is created by the arteries as they resist the blood flow. Blood pressure is the result of these two forces.

#### Is my blood pressure normal?

See the following blood pressure classification chart released by the WHO (World Health Organization) for evaluation of your blood pressure level.

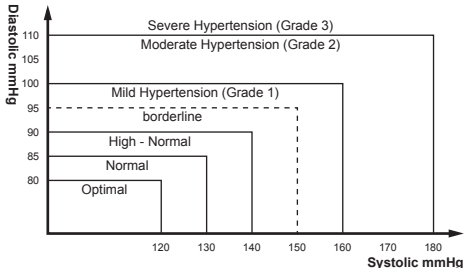## Information you should know before operating the unit

Classification of blood pressure monitor	Systolic mmHg	Diastolic mmHg	Colour indicator
Optimal	< 120	< 80	green
Normal	120 - 129	80 - 84	green
High - Normal	130 - 139	85 - 89	green
Grade 1 Hypertension	140 - 159	90 - 99	yellow
Grade 2 Hypertension	160 - 179	100 - 109	orange
Grade 3 Hypertension	$\geq 180$	$\geq 110$	red

### What are systolic and diastolic pressures?


Systolic pressure is the highest pressure at the height of the heart's contraction. Diastolic pressure is the lowest pressure when the heart is resting.

### What about low blood pressure?

In general, a lower blood pressure reading is better unless it causes unpleasant symptoms such as fainting and/ or lightheadedness.

### Fluctuation and variation in blood pressure

The following chart shows possible blood pressure fluctuations during a 24-hour period.

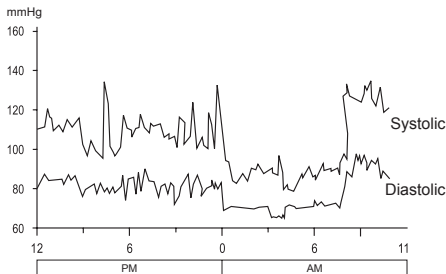EN

## Information you should know before operating the unit

The following factors will influence blood pressure measurement results and cause variations:

- Bathing
- Drinking alcohol
- Moving
- Meals
- Thoughts
- Conversation
- Exercise
- Stress
- Temperature change
- Smoking etc.

### Does it matter whether the measurement is taken on the left or the right arm?


Measurements can be taken on the left or right arm. However, further measurements should always be taken on the same arm - preferably on the arm on which higher blood pressure is measured.

## Description of the device

### Information about the Device

EN

Product label  
with serial  
number

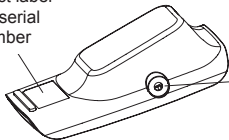
#### Content:

1. Blood pressure monitor (GT-1115)
2. Cuff (Type BF applied part)  
22 cm - 42 cm
3. User manual
4. USB cable (5 V  $\overline{\text{---}}$  1 A)
5. Storage bag


**Note:** Only use the cuffs supplied by Geratherm® specifically for this device.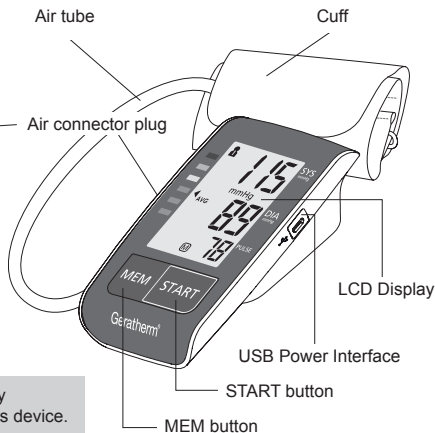

## About the unit

### Explanation of the Display

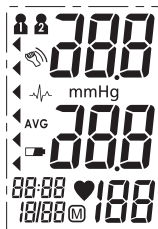Symbol	Description	Explanation
<b>SYS</b>	Systolic blood pressure	Higher BP value in mmHg
<b>DIA</b>	Diastolic blood pressure	Lower BP value in mmHg
<b>PULSE</b>	Pulse	Pulse in beats per minute
	Movement detector	Movement will result in inaccurate
	Memory	The displayed measurement values are taken from the memory
<b>AVG</b>	Average value	The average value of the latest 3 blood pressure measurement results
	Low battery	Battery is low and needs to be charged
	Arrhythmia	Irregular heartbeat detection
	WHO classification	WHO evaluation of the measured values
	Current date and time	Month/Day, Hour/Minute
	Heartbeat	Heartbeat detection during the measurement
	User 1	Start measurement and save the measuring results for User 1
	User 2	Start measurement and save the measuring results for User 2

EN

## How to operate the unit

### Power Supply and Charge Power

1. The battery of Geratherm® active control+ is a built-in rechargeable lithium-polymer battery with an electrical charge of 1000 mAh.
2. Please use a power adapter with a USB connection (not included) or another source of power with a USB connection and the enclosed USB cable for charging the rechargeable battery, as shown in the following illustrations

Method 1

Method 2

**Note: Optional Adapter**

(Please use any authorized adapter)

Input: AC 100 - 240 V ~50/60 Hz 0,2 A Max

Output: 5 V  $\overline{\text{---}}$  1000 mA

**Power needs to be charged under the following circumstances:**


-  displays on the LCD

## How to operate the unit





### Notes

If you switch on the blood pressure monitor and the LCD display flashes "  ". This means that the battery charge is low. Please recharge it.

You can, however, still carry out the measurement.

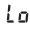If the LCD display "  " appears, this means that the battery charge is too low. The blood pressure monitor will switch off automatically and you must recharge it immediately.

During recharging, the LCD display flashes and shows the following symbols:

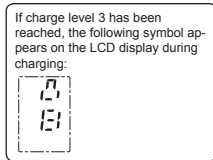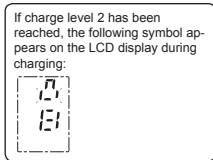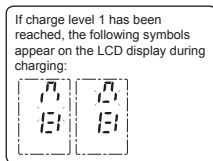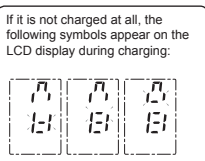### Notes on rechargeable battery

- The rechargeable battery of the Geratherm® active control+ is a built-in rechargeable lithium-polymer battery. Please do not attempt, nor allow unauthorised maintenance personnel to take the blood pressure monitor apart or to force it open.
- In the context of normal use, the battery can be recharged approximately 300 times. If the battery does not load or the device cannot be used normally, please consult the retailer. If you measure your blood pressure three times per day, the device can be used for up to 40 days without being charged.
- You cannot use the blood pressure monitor while it is charging. Once the device is fully charged, please unplug it promptly.
- Use only the authorised USB cable and power adaptor for the Geratherm® (model: BLJ-06L050100U-V), to charge it. Use the special power adaptor and USB cable that comply with CE safety standards to achieve the best performance and protect your monitor.
- While it is charging, do not touch the charger connection and the patient simultaneously.
- Do not attempt to replace the rechargeable battery in your blood pressure monitor. The battery is built-in and not replaceable.
- Avoid charging the blood pressure monitor in extremely high or low temperatures.
- Only charge the battery according to the instructions provided with the blood pressure monitor. Store and use the blood pressure monitor in a cool, dry and well-ventilated place. Keep it away from fire and heat sources, otherwise the battery could explode!

## How to operate the unit


### Activate your blood pressure monitor

Your blood pressure monitor is activated when the date and time are set.

### Setting Date and Time

It is important to set the clock before using your blood pressure monitor, so that a time stamp can be assigned to each record that is stored in the memory. (time format: 24 H)

1. When the monitor is off, hold pressing "MEM" button about 3 seconds to enter the mode for year setting.

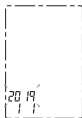
2. Press "START" button to change the year. Each press will increase the numeral by one in a cycling manner.

3. When you get the right year, press "MEM" button to set down and turn to next step.


4. Repeat steps 2 and 3 to set the [MONTH] and [DAY].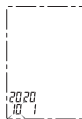

EN

## How to operate the unit

- Repeat steps 2 and 3 to set the [HOUR] and [MINUTE].

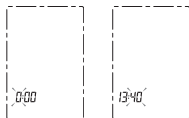- After the [MINUTE] is set, the LCD will display all the settings you have done and then turn off.

### Select User

- When the monitor is off, press and hold "START" button to enter user setting mode. The user ID will blink.

- Then press "MEM" button to select the user ID between User 1 and User 2.


- After selecting the suitable user ID, press "START" button to confirm. Then the LCD will turn off.

### Attaching the pressure cuff

- Before taking a blood pressure measurement, remove any jewellery, such as watches and bracelets, on the arm that is to be used for measuring.

**Note:** If your doctor has diagnosed poor circulation in your left arm, use the right arm.

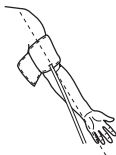
## How to operate the unit

2. In order to be able to wrap the cuff around the arm, push or roll your sleeve up to expose the bare skin. Make sure your sleeve is not too tight.

Hold your arm with the palm of your hand facing upwards and attach the cuff to the upper arm over the main artery.

The air tube must be on the inner side of the arm in line with the little finger. On the cuff there is also a printed label that you can use for guidance.

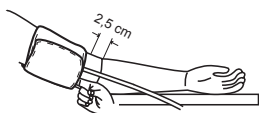**Note:** You can find the main artery by pressing two fingers approximately 2 centimetres above the bend of the elbow on the inner side of the arm. The point where the pulse can be strongly felt is the location of the main artery.

3. The cuff should be fitted snugly and comfortably but should not constrict the arm. It should be possible to insert a finger between the cuff and the arm.

### Posture during measurement

1. Sit comfortably and lay your arm with the cuff on a flat surface. Lay your elbow on the table so that the cuff is level with your heart. The palm of your hand should face upwards. Sit upright and take 5 or 6 deep breaths.

1. Measure and record blood pressure at the same time every day to establish your blood pressure pattern.

EN

### Helpful tips for patients, particularly patients with hypertension:


- Rest for five minutes before measuring.
- Wait at least three minutes between measurements. This will enable your circulation to recover.
- Take the measurement in a quiet room.
- Try to relax during measurement; do not move or speak.
- Attach the cuff at a position level with your heart.
- Sit comfortably. Do not cross your legs but place your feet flat on the floor, and lean against the back of the chair.
- In order to ensure comparability of measurements, try to measure under similar conditions each time. For example, measure at the same time each day, using the same arm, or following the instructions of your doctor.

## Functions


### Measuring blood pressure

1. When the monitor is off, press the "START" button to turn on the monitor, the measurement will be completed automatically.


LCD display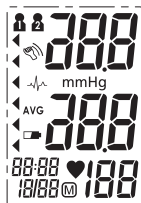

0 – value is displayed

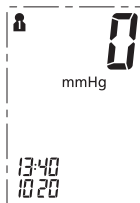Inflating and measuring

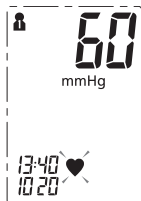Display and save the measurement results

EN

2. As soon as measurement is completed, the blood pressure and pulse readings appear on the display. Press the "START" button to switch the unit off. Otherwise, the unit will switch off automatically after one minute.

Both User 1 and User 2 have 60 memory slots.

## Functions

### Irregular heartbeat detector


An irregular heartbeat is detected when a heartbeat rhythm varies while the unit is measuring the systolic and diastolic blood pressure. During each measurement, this equipment records the heartbeat intervals and works out the standard deviation. If major pulse irregularities occur, the arrhythmia icon appears on the display along with the blood pressure readings.

### CAUTION

The appearance of the arrhythmia icon indicates that a pulse irregularity consistent with an irregular heart-beat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

### Retrieving stored readings

1. In order to retrieve the latest readings, press the "MEM" button and release it. The display will show the average value (AVG) of the last three readings. (Note: If fewer than three readings are available, the latest reading is shown)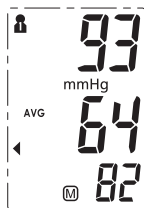


EN

## Functions


- By pressing again on the “MEM” button, all stored readings can be retrieved. The number and date of the stored reading can be seen in the bottom left-hand corner. The stored reading appears alternately with the time when the reading was taken.

⚠ The most recent record (1) is shown first. Each new measurement is assigned to the first (1) record. All other records are pushed back one digit (e.g., 2 becomes 3, and so on), and the last record is dropped from the list.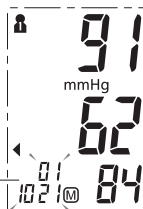

EN

- In order to retrieve the readings of another user, select that user in advance. See section on “Selecting a user”. To retrieve the stored readings, press the “MEM” button and release it.

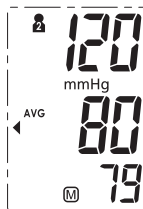- Press “START” button to power off, otherwise it will turn off within 1 minute.

## Functions

### Deleting readings

1. Press and hold the "MEM" button for about 3 seconds when the unit is already in memory mode.

The user and "dEL ALL" (delete all) appears on the display.

2. Press the "MEM" button to confirm the deletion process. "dEL dONE" (deletion done) appears on the display.

EN

3. To check whether the deletion process was successful, press the "MEM" button. "0" should appear on the display.

**Note:**

To leave deletion mode without deleting readings, press the "START" button.

 Care and maintenance

Do not drop the unit.

Avoid thinner, benzene, and other harsh cleaners.

Do not modify or disassemble the unit or the arm cuff.

Keep the unit in a suitable place. Avoid high temperature, direct sunlight, high moisture, and dust.

EN

Do not twist the cuff.

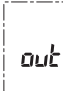
Do not press the "START" button if the cuff has not been properly wrapped around the arm.

Use a cloth moistened with water or neutral detergent to clean the body of the unit and then wipe it dry.

## Error messages

Problem	Display symbol	Cause	Solution
No power	Nothing appears on the display	Battery charge too low	Charge battery
Low battery power	→  Indicator	Battery charge low	Charge battery
No power	Nothing appears on the display	Adapter / charging cable is not connected correctly	Check adapter and charging cable and plug in correctly
No display	No display when buttons pressed	Unit is not activated	Press and hold down „MEM“ button to activate the unit
Error messages	E 01	The cuff is not properly fastened or pressure in the cuff is too high	Rest for a short period of time, re-fasten the cuff and measure the blood pressure again
	E 02	The unit has detected movement or pulse rate is too low	Movement can influence the measurement. Relax for a moment and then measure again
	E 03	No pulse signal is detected	Loosen clothing around the arm and measure again
	E 04	The measuring process was not successful	Rest for a short time and retake the measurement
	EExx	A calibration error has occurred	Repeat the measurement. If the problem persists, contact your dealer or our customer service department for further support
		Measured values are outside the measurement range	Relax for a moment. Re-fasten the cuff and measure again. If the problem re-occurs, consult your physician

EN

## Specifications

Model-No.	GT-1115
Power supply	3,7 V 1000 mAH built-in rechargeable lithium-polymer battery
Display mode	Digital LCD Display V.A. 44 mm x 64 mm
Measurement mode	Oscillographic testing mode
Measurement range	Rated cuff pressure: 0 mmHg ~ 299 mmHg Measurement pressure: SYS: 60 mmHg ~ 230 mmHg DIA: 40 mmHg ~ 130 mmHg Pulse value: (40 - 199) beat/minute
Accuracy	Pressure: +/-3 mmHg Pulse value: +/- 5 %
Operating condition	A temperature range between: +5 °C and +40 °C A relative air humidity range of: ≤85 % non-condensing but not requiring water vapour partial pressure of more than 50 hPa An atmospheric pressure range of: 700 hPa to 1060 hPa
Storage and transportation condition	Temperature: -20 °C to +60 °C A relative air humidity range of ≤ 93 %, non-condensing, with water vapour pressure up to 50 hPa
Cuff size	22 cm ~ 42 cm
Net Weight	approx. 177 g excluding cuff
External dimensions	approx. 131 mm x 78 mm x 42 mm
Attachment	USB cable, user manual, storage bag
Degree of protection	Type BF applied part
Protection against ingress of water	IP21, this means: The device is protected against penetration by solid objects >= 12.5 mm in diameter and against the ingress of water in the form of vertically falling drops.
Device classification	Battery mode: ME device with built-in lithium-polymer rechargeable battery
Software Version	A01

Subject to changes in the interests of technical progress.

## Quality standard

Geratherm Medical AG is certified in accordance with DIN EN ISO 13485 and fulfils the requirements of Council Directive 93/42/EEC. It is therefore authorised to apply the label C€0197 (Notified Body: TÜV Rheinland LGA Products GmbH).

## Symbol index

	Follow the instructions for use
	Keep dry
	Store at relative humidity levels between 0 % and 93 % R.H.
	Caution! Read the user manual
	The device must not be disposed of with household waste
	Direct current

	Type BF Equipment
	Manufacturer
	Store between -20 °C and +60 °C
	(YYMMXXX; Year/Month/Serial number)
	Manufacturing date

EN

## Information on electromagnetic compatibility (EMC)

Electronic devices such as PCs and mobile phones can lead to the exposure of medical devices in operation to electromagnetic interference from other devices. This can lead to malfunction of the medical device and create a potentially unsafe situation.

Medical devices should also not interfere with any other devices.

The EN 60601-1-2 standard regulates the requirements for EMC (electromagnetic compatibility) and defines the levels of immunity to electromagnetic interference and the maximum electromagnetic emission levels for medical devices.

This blood pressure monitor, which is manufactured by Geratherm Medical AG, complies with the EN 60601-1-2 standard in relation to both immunity and emissions.

However, special precautions should be observed:

Please only use the device indoors and not in the vicinity of mobile phones or microwave ovens.

In the case of devices whose maximum power exceeds 2 W, the minimum distance from your blood pressure monitor should be 3.3 metres.

EN

### Warning:

The use of accessories, transducers and cables other than those specified or provided by the manufacturer of this device may result in increased electromagnetic emissions or reduced electromagnetic immunity of the device and may cause it to malfunction.

Geratherm Medical AG  
Fahrenheitstraße 1  
99331 Geratal  
Germany  
[www.geratherm.com](http://www.geratherm.com)

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