Deutsch

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Click on

your language

English

INSTRUCTIONS FOR USE..... 3

Polski

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Français

MODE D'EMPLOI..... 4

Česky

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Italiano

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Slovensky

NÁVOD NA POUŽÍVANIE..... 16

Español

INSTRUCCIONES DE USO..... 6

Lietuvių

NAUDOJIMO INSTRUKCIJA..... 17

Português

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Latviešu

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Nederlands

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Eesti

KASUTUSJUHEND..... 19

Dansk

BRUGSANVISNING..... 9

Hrvatski

UPUTE ZA UPORABU..... 20

Norsk

BRUKSANVISNING..... 10

Slovenščina

NAVODILA ZA UPORABO..... 21

Svenska

BRUKSANVISNING..... 11

Srpski

UPUTSTVO ZA UPOTREBU..... 22

Suomi

KÄYTTÖOHJE..... 12

Magyar

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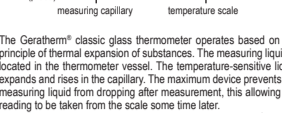
Product overview**Packaging contents**

1x clinical glass thermometer
 1x protective case
 1x instructions for use
 optional according to product configuration:
 Geratherm® classic: 1x reset tool
 Geratherm® classic plus: 1x storage case with magnifying lens
 (2 parts)

Product description

The Geratherm® classic glass thermometer operates based on the principle of thermal expansion of substances. The measuring liquid is located in the thermometer vessel. The temperature-sensitive liquid expands and rises in the capillary. The maximum device prevents the measuring liquid from dropping after measurement, this allowing the reading to be taken from the scale some time later.

Since 1992, we have been using the harmless Galinstan® as the measuring liquid. Without toxic mercury.

Intended purpose

Clinical glass thermometer intended for the rectal, oral, or axillary measurement of body temperature.

Indications

- Suspected fever or elevated body temperature (hyperthermia)
- Monitoring body temperature in cases of illness accompanied by fever (temperature monitoring)

Contraindications

The Thermometer must not be used under the following conditions:

- determining lowered body temperature (hypothermia)

Patient target group

Within the specified measuring range and the specified measuring sites, this thermometer can be used for all patients regardless of age, body weight and gender.

Intended users

This thermometer is intended for use by adults or for use by children and adolescents under the supervision of an adult in private and professional healthcare environment.

Limitations

Not suitable for determining basal body temperature for cycle monitoring.

⚠️ Precautions, warnings and residual risks

Disregarding the intended purpose and lacking knowledge regarding the thermometer's usage can lead to inaccurate measurements and misdiagnosis.

- Use the thermometer only according to these instructions.
- Pay attention to all information on the thermometer and its packaging.
- Keep the instructions for future users and future applications.

Using the thermometer does not replace a doctor's diagnosis.

- If you have persistent high body temperature or fever, consult your doctor!

The thermometer used, the measuring site, and physical activity influence the measurement result.

- Compare only readings obtained at the same measuring site and with the same thermometer.
- Do not take temperature measurements within 30 minutes after exercising.

Habits in daily life can lead to incorrect measurements.

- Do not consume hot or cold drinks before taking your temperature.
- Avoid taking your temperature immediately after eating, bathing, or smoking.

Excessive force when inserting the thermometer can cause injury to the surrounding tissue.

- Do not force insertion and stop if you encounter resistance.

This thermometer is made of glass, which can break and pose a risk of injury.

- Keep the thermometer out of reach of children.
- Temperature measurement of children should only be done by an adult.
- Do not bite down on the thermometer during oral measurement.
- Do not use automated cleaning or disinfection for the thermometer.

Germs, such as bacteria and viruses, can be transmitted from one measuring site to another and from patient to patient during use.

- Clean and disinfect the thermometer before and after each measurement to avoid infections.
- If you have a wound at your preferred measuring site, choose a different measuring site.

Improper transport, storage, or handling can damage the thermometer.

- Always keep the thermometer away from direct sunlight, temperatures below -15 °C (5 °F) and above 42 °C (107.6 °F), and dirt.
- Protect the thermometer from impacts and hits.
- Always store the thermometer in its designated protective case.

Defective thermometers yield inaccurate measurements.

- Check your thermometer for visible damage before each use.
- Do not use the thermometer if it is damaged.

If you are unsure about the measurement obtained, repeat the measurement.

- If necessary, consult a doctor.

The Geratherm® classic should not be used by:

- Persons who are mentally or physically incapable of using the thermometer in accordance with these instructions for use.

In the event of serious incidents related to the thermometer, these should be reported to Geratherm Medical GERMANY GmbH and the competent authority of your Member State.

Notes for Measuring site

Rectal measurement is recommended for children up to 4 years of age. For children aged 4 years and older and adults, rectal or oral measurement should be preferred.

Application**Execute a measurement**

1. Remove the thermometer from its protective case.
2. Clean and disinfect the product before use according to the cleaning and disinfection instructions in chapter "Cleaning & Disinfection".
3. Check if the measuring liquid is below 36 °C (96.8 °F). If not, use the provided reset tool to lower the measuring liquid first.

Before each new measurement, you must reset the measuring liquid of the thermometer. To do this, use the provided:

- a) reset tool (Geratherm® classic) or
- b) insert of the storage case with magnifying lens (Geratherm® classic plus).

Note: Ensure that you have sufficient distance from objects and furniture to prevent damage to the thermometer.

Insert the thermometer into the reset tool. Ensure that the thermometer is securely fixed in the slots. The tip of the thermometer protrudes 2 cm below. Hold the reset tool between your thumb and index finger on the flat grip surface, with the measuring tip pointing towards the floor. Begin shaking with the thermometer with your wrist (left-right) until the measuring fluid is lowered below 36 °C (96.8 °F).

4. Position the thermometer at the chosen measuring site.

Rectal measurement – temperature measurement in the anus. Apply a little lubricant (e.g. vaseline or caring oil) to the measuring tip to make insertion easier. Then carefully insert the tip of the thermometer approx. 2 cm to 3 cm into the anus (rectum).

Oral measurement - Temperature measurement in the mouth. Place the measuring tip in one of the two heat pockets under the tongue, to the left or right of the root of the tongue. Keep your mouth closed during the oral temperature measurement and continue to breathe evenly through your nose to establish a thermal equilibrium.

Heat pockets under the tongue

Axillary measurement - Temperature measurement under the armpit. Dry the armpit with a towel and place the measuring tip in the armpit. Close the arm tightly. Keep the armpit closed during the axillary temperature measurement to achieve thermal equilibrium.

5. After a minimum of 4 minutes, the thermometer will have taken the body temperature.
6. The measuring liquid in the measuring capillary now displays the reading, which can be read from the scale.

Adjust the angle of the thermometer slightly to make reading the measuring liquid against the blue background of the capillary easier. Be aware of good light conditions during read out results.

Note for the product configuration storage case with magnifying lens: To read the measured value of the thermometer even easier, place the thermometer in the white insert and slide it into the storage case. The values appear larger due to the magnifying effect.

7. Clean the thermometer again according to chapter "Cleaning & Disinfection".
8. Reset the measuring fluid afterward as described in step 3.
9. Place the thermometer back into the protective case with the tip facing down for storage. Optionally, you can also store the thermometer in the storage case with magnifying lens. Make sure that the tip of the thermometer does not protrude from the front end.

Cleaning & Disinfection

The thermometer has a hermetically sealed glass sheath and is waterproof.

Ensure that all surfaces are accessible for cleaning and disinfection. Clean the thermometer before performing disinfection. Remove any coarse contaminants and, if necessary, residual lubricants beforehand using a dry and clean single-use cloth. For cleaning, dampen a clean, soft cloth with a soap solution (maximum 42 °C / 107.6 °F) and thoroughly clean the entire surface of the thermometer. To disinfect the thermometer, dampen a clean, soft cloth with alcohol-based disinfectant solution (e.g. 70% isopropyl alcohol). Wipe the thermometer towards the tip and allow the disinfectant to take effect according to the manufacturer's instructions.

To remove residues of cleaning or disinfectant agents, dampen a new clean, soft single-use cloth with pure water and wipe it over the thermometer several times. If there is still visible contamination after cleaning, repeat the steps listed above.

Disposal

The packaging and instructions for use are made of cardboard and paper and can be disposed of according to your local waste disposal guidelines. In the event of thermometer breakage, the spilled liquid can be removed using a brush, piece of cellulose, or cloth dampened with soap solution. The remnants can be disposed of in household waste, adhering to your local waste disposal guidelines. Caution: Danger of cuts!

Guarantee

The manufacturer provides a 10-year guarantee for manufacturer-caused defects from the date of purchase on this thermometer. Damages caused by improper handling (e.g. breakage) are not covered by the guarantee. Claims can only be considered with presentation of the purchase receipt.

Technical data

Operating conditions	-15 °C (5 °F) to $+35\text{ °C}$ (95 °F)
Minimum measuring range	$+35,5\text{ °C}$ ($95,9\text{ °F}$) to $+42\text{ °C}$ ($107,6\text{ °F}$)
Measurement resolution	$0,1\text{ °C}$ ($0,18\text{ °F}$)
Measurement accuracy	$-0,15\text{ °C}$ / $+0,1\text{ °C}$ ($-0,27\text{ °F}$ / $+0,18\text{ °F}$)
Storage conditions	-15 °C (5 °F) to $+42\text{ °C}$ ($107,6\text{ °F}$)
Quality guarantee	The thermometers are manufactured and tested in accordance with the requirements of EN 12470-1 Clinical thermometers – Part 1: Metallic liquid-in-glass thermometers with maximum device.

Symbol Index

	Manufacturer		Catalogue number
	Follow instructions for use		Model number
	Medical device		Batch code
	Unique device identifier		Temperature limit
	Fragile, handle with care		Keep away from sunlight
	Caution!		Authorised Swiss representative

GT-1000

GT001001E

Rev.05_2024-06-26

Contact information

For more information about our devices and a digital version of the instructions for use, please visit our website www.geratherm.com.

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